

Getting Started with the Versah[®] Guided Surgery System



Versah[®] Certified Labs

A full list of certified labs available at versah.com/guided-surgery-system/

Determine the Following:

- Tooth #
- Implant Diameter

Plan the Case:

- Take a CBCT Scan, Dicom File, and impressions of the site and the opposing jaw.
- Contact a Versah[®] Certified Lab to begin the process of printing a custom C-Guide[®] Sleeve.

Minnesota Dental Lab

www.minnesotadentallab.com

Ottawa Dental Laboratory

www.ottawadentallab.com

Van Hook Dental Studio

www.vhdental.com

3D Diagnostix Inc. (3DDX)

www.3ddx.com

Versah[®] Guided Surgery System

Price Breakdown

Item	Unit Price
G-Stop [®] Vertical Gauge ...	\$18 each
G-Stop [®] Key	\$49 each
C-Guide [®] Sleeve.....	\$19 each
G-Stop [®] Holder	\$170



I Use CEREC

- Set up your software to produce the guide with a hole that matches the Versah[®] C-Guide[®] Sleeve outer-diameter.
- Create buccal window to match the Versah[®] C-Guide[®] Sleeve opening.
- Order the C-Guide[®] Sleeve and the compatible G-Stop[®] from Versah[®].

I Print My Own Guide

- Set up your software to produce the guide with a hole that matches the Versah[®] C-Guide[®] Sleeve outer-diameter.
- Create buccal window to match the Versah[®] C-Guide[®] Sleeve opening.
- Order the C-Guide[®] Sleeve and the compatible G-Stop[®] from Versah[®].

I'd Like to Use My Lab

- Versah[®] is committed to certify your dental lab to produce the Universal C-Guide[®] Surgical Guide.

Please call us at:
517-796-3932



Versah[®] Guided Surgery System Compatible Software



Sizing the Versah[®] Guided Surgery System

G-Stop[®] Vertical Gauge




Available Sizes

Densah[®] Bur Compatibility

G-Stop[®] Vertical Gauge + Key

C-Guide[®]

S
Placement up to 3.25 mm Implant

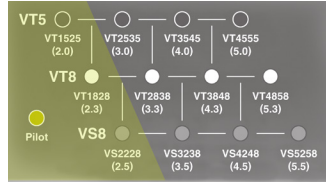

OD 4.2 mm

ID 4.40 mm




6.25 mm

1 mm

OD 5.3 mm

M
Placement up to 4.3 mm Implant

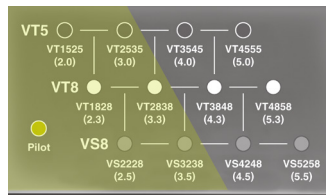

OD 5 mm

ID 5.20 mm




6.25 mm

1 mm

OD 6.1 mm

L
Placement up to 5.3 mm Implant

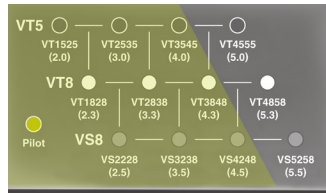

OD 6 mm

ID 6.20 mm




6.25 mm

1 mm

OD 7.1 mm

XL
Placement up to 6.2 mm Implant

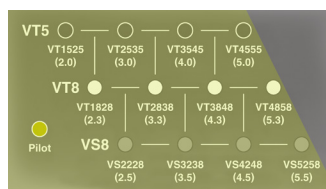

OD 7 mm

ID 7.20 mm



6.25 mm

1 mm

OD 8.1 mm

Available G-Stop[®] Drilling Depths

3 mm | 5 mm | 8 mm | 10 mm | 11.5 mm | 13 mm | 15 mm



C-Guide[®] Sleeve



WWW.VERSAH.COM

