

Densah™ Bur Technology At Work:

Case: 64 year old male patient presented with missing maxillary right canine



1 - Significant loss in alveolar ridge due to buccal plate resorption



2 - Alveolar ridge width after flap reflection measuring 3 mm



3 - Densah Bur 1828 osseodensification mode 1200 RPM (non-cutting direction)



4 - Densah Bur 2838 Osseodensification mode 1200 RPM (Non-cutting direction)



5 - Osseodensification facilitated ridge expansion with compaction Autografting



6 - 4.2 mm fixture placed. Evidence of ridge expansion with no fenestration



7 - 3 wks healing with apparent ridge expansion



8 - Final Restoration with adequate buccal ridge anatomy

To watch a video of this clinical procedure and others,
go to www.versah.com and click on "Clinical Video."
Choose from a library of videos showing Densah Bur Technology in action.

Densah™ Bur Technology Provides Your Implant Practice With Versatile Osteotomy Preparation

Clockwise: Precisely Cuts Dense Bone

Counterclockwise: Osseodensification of Soft to Medium Bone

- Narrow Ridge Expansion • Sinus Lift Procedures • Implant Placement in All Areas of the Mouth

Bring Densah Bur Technology Into Your Implant Practice

Ordering Information

Densah Bur Kits			
Catalog #	Description	Price	Includes
VDBK	Densah Bur Kit Complete	\$1478.00	All Densah Burs Below and VBLK
VT5	Densah Bur Kit 1525-4555	\$526.00	VT1525, VT2535, VT3545, VT4555 and VBLK
VT8	Densah Bur Kit 1828-4858	\$526.00	VT1828, VT2838, VT3848, VT4858 and VBLK
VS8	Densah Bur Kit 2228-5258	\$526.00	VS2228, VS3238, VS4248, VS5258 and VBLK

Individual Densah Bur Items			
Catalog #	Description	Price	Available Sizes
VT1525	Densah Bur	\$119.00	1.5mm to 2.5mm Tapered Implants
VT2535	Densah Bur	\$119.00	2.5mm to 3.5mm Tapered Implants
VT3545	Densah Bur	\$119.00	3.5mm to 4.5mm Tapered Implants
VT4555	Densah Bur	\$119.00	4.5mm to 5.5mm Tapered Implants
VT1828	Densah Bur	\$119.00	1.8mm to 2.8mm Tapered Implants
VT2838	Densah Bur	\$119.00	2.8mm to 3.8mm Tapered Implants
VT3848	Densah Bur	\$119.00	3.8mm to 4.8mm Tapered Implants
VT4858	Densah Bur	\$119.00	4.8mm to 5.8mm Tapered Implants
VS2228	Densah Bur	\$119.00	2.2mm to 2.8mm Straight Implants
VS3238	Densah Bur	\$119.00	3.2mm to 3.8mm Straight Implants
VS4248	Densah Bur	\$119.00	4.2mm to 4.8mm Straight Implants
VS5258	Densah Bur	\$119.00	5.2mm to 5.8mm Straight Implants
VBLK	Densah Bur	\$50.00	Block

Stop drilling away healthy bone
and increase implant stability with Osseodensification.

To order the newest innovation in implant dentistry,
call a Versah™ Customer Service Professional
at 844-711-5585 or visit www.versah.com.



2500 West Argyle Street | Suite 300 | Jackson, MI 49202
P: 517-796-3932 | Toll Free: 844-711-5585 | Fax: 844-571-4870

