

Densah™ Bur Technology At Work:

Case: 64 year old male patient presented with missing maxillary right canine



1 - Significant loss in alveolar ridge due to buccal plate resorption



2 - Alveolar ridge width after flap reflection measuring 3 mm



3 - Densah Bur 1828 osseodensification mode 1200 RPM (non-cutting direction)



4 - Densah Bur 2838 Osseodensification mode 1200 RPM (Non-cutting direction)



5 - Osseodensification facilitated ridge expansion with compaction Autografting



6 - 4.2 mm fixture placed. Evidence of ridge expansion with no fenestration



7 - 3 wks healing with apparent ridge expansion



8 - Final Restoration with adequate buccal ridge anatomy

To watch a video of this clinical procedure and others, go to www.versah.com and click on "Clinical Video." Choose from a library of videos showing Densah Bur Technology in action.